



Distributed by AD Surgical

www.AD-Surgical.com

888.841.8481

AD Surgical, Inc
1296 KIFER RD, STE 608,
SUNNYVALE, CA 94086, U.S.A

Read this package insert completely before use and comply with the instructions for use.

HealiAid[®] Collagen Wound Dressing

DOH-MD-No. 003197

GRAPHICAL SYMBOLS

	Single use only		Catalog number
	Read instructions before use		Lot number
	Sterilization by irradiation		Store at room temperature
	Do not re-sterilize		Keep away from sunlight
	Use by expiry date		Do not use if package is damaged
	Manufacturer		

Description

HealiAid[®] Collagen Wound Dressing is a white, porous, pliable and absorbable collagen wound dressing. HealiAid[®] Collagen Wound Dressing is fabricated by native fibrous collagen matrix purified from Achilles bovine tendons. HealiAid[®] Collagen Wound Dressing is pliable and can be applied easily to bleeding wounds. The product is supplied in a sterile, non-pyrogenic package, and is indicated for single use only.

Indications

HealiAid[®] Collagen Wound Dressing is a white, pliable sheet material which consists native fibrous collagen matrix purified from bovine Achilles tendon. HealiAid[®] Collagen Wound Dressing is flexible for easily application to wound. HealiAid[®] Collagen Wound Dressing has better bio-compatibility with positive cosmetic effects, and can better stimulate wound healing when compared to other wound dressing products.

HealiAid[®] Collagen Wound Dressing can be used in the following treatments:

- Periodontal Surgery
- Implantology
- Sinus Elevation
- Defect Surgery
- Stop Bleeding after an Odontectomy.

Instructions for using HealiAid[®] Collagen Wound Dressing

1. After dental extraction, press HealiAid[®] Collagen Wound Dressing on the wound site to stop bleeding for two to five minutes.
2. Fill wound site with HealiAid[®] Collagen Wound Dressing.
3. Suture the wound site.

Precaution

- HealiAid[®] Collagen Wound Dressing should not be applied to infected area of a wound site.
- Clean the contaminated area carefully before applying HealiAid[®] Collagen Wound Dressing onto the wound site to prevent infection.
- Even though HealiAid[®] Collagen Wound Dressing promotes blood clotting, HealiAid[®] Collagen Wound Dressing is not intended to apply to patients with abnormal blood clotting.
- The safety of HealiAid[®] Collagen Wound Dressing on pregnant women is not clinically evaluated. Surgeon should evaluate the application of HealiAid[®] Collagen Wound Dressing in pregnant women.
- Patients with sever allergy history or hypersensitivity to bovine collagen not recommended to use HealiAid[®] Collagen Wound Dressing.

- HealiAid[®] Collagen Wound Dressing is contraindicated to patients undergoing desensitization injections to meat products, as these injections may contain bovine collagen.
- Check the expiration date of each individual product before use. Do not use expired products.
- Do not use HealiAid[®] Collagen Wound Dressing when packaging is damaged, as sterility of the contents cannot be assured.
- HealiAid[®] Collagen Wound Dressing is limited to single use only. Do not re-sterilize or re-use HealiAid[®] Collagen Wound Dressing.

Specifications

HealiAid[®] Collagen Wound Dressing is a collagen wound dressing. Each product is well-packed in sterile (γ-irradiation) condition. The specifications of HealiAid[®] Collagen Wound Dressing products are listed below:

Product type	Specification
--------------	---------------

HealiAid[®]-Sheet

SA204030	20mm x 40mm x 3.0mm
SA505030	50mm x 50mm x 3.0mm
SA100100	100mm x 100mm x 3.0mm
SA152005	15mm x 20mm x 0.5mm
SA203005	20mm x 30mm x 0.5mm
SA253005	25mm x 30mm x 0.5mm
SA304005	30mm x 40mm x 0.5mm

HealiAid[®]-Plug

SA001020	ø10mm x 20mm
SA001520	ø15mm x 20mm

HealiAid[®]-Particle

SA-G01	1cc
SA-G02	2cc
SA-G05	5cc
SA-G10	10cc
SA-G20	20cc

Storage:

HealiAid[®] should be stored at room temperature in a dry place. Do not freeze or expose HealiAid[®] to extreme environmental conditions.

Warning

HealiAid[®] Collagen Wound Dressing cannot be autoclaved because the collagen fibrous structure as well as the packaging will be damaged. HealiAid[®] Collagen Wound Dressing should not be re-sterilized. Residual portions of HealiAid[®] Collagen Wound Dressing should be discarded after use.

HealiAid[®] Collagen Wound Dressing contains bovine collagen and must not be use to patients with a history of allergy to any bovine collagen products. Hypersensitivity reactions reported when using bovine collagen products including erythema, swelling, induration, and/or urticaria.

HealiAid[®] Collagen Wound Dressing should be stored at room temperature in a dry place. Do not expose HealiAid[®] Collagen Wound Dressing to extreme environmental conditions. Do not use after expiration date.

MBI MAXIGEN BIOTECH INC.

MAXIGEN BIOTECH INC.
No.88, Keji 1st Rd., Guishan District, Taoyuan City 33383, Taiwan
TEL : +886-3-3287-222 FAX : +886-3-3287-333
E-MAIL : service@mbi-bio.com

21H2B0016